

OLYMPUS[®]

Your Vision, Our Future

OLYMPUS
Endo**Therapy**
BEYOND THE SCOPE

Single Use 2-Lumen Sphincterotome V


CleverCut^{2V}

Single Use 3-Lumen Sphincterotome V

CleverCut^{3V}



Exceptional Cutting Performance and Easy, Fast Exchange Capability for Enhanced Efficiency in ERCP Sphincterotomy

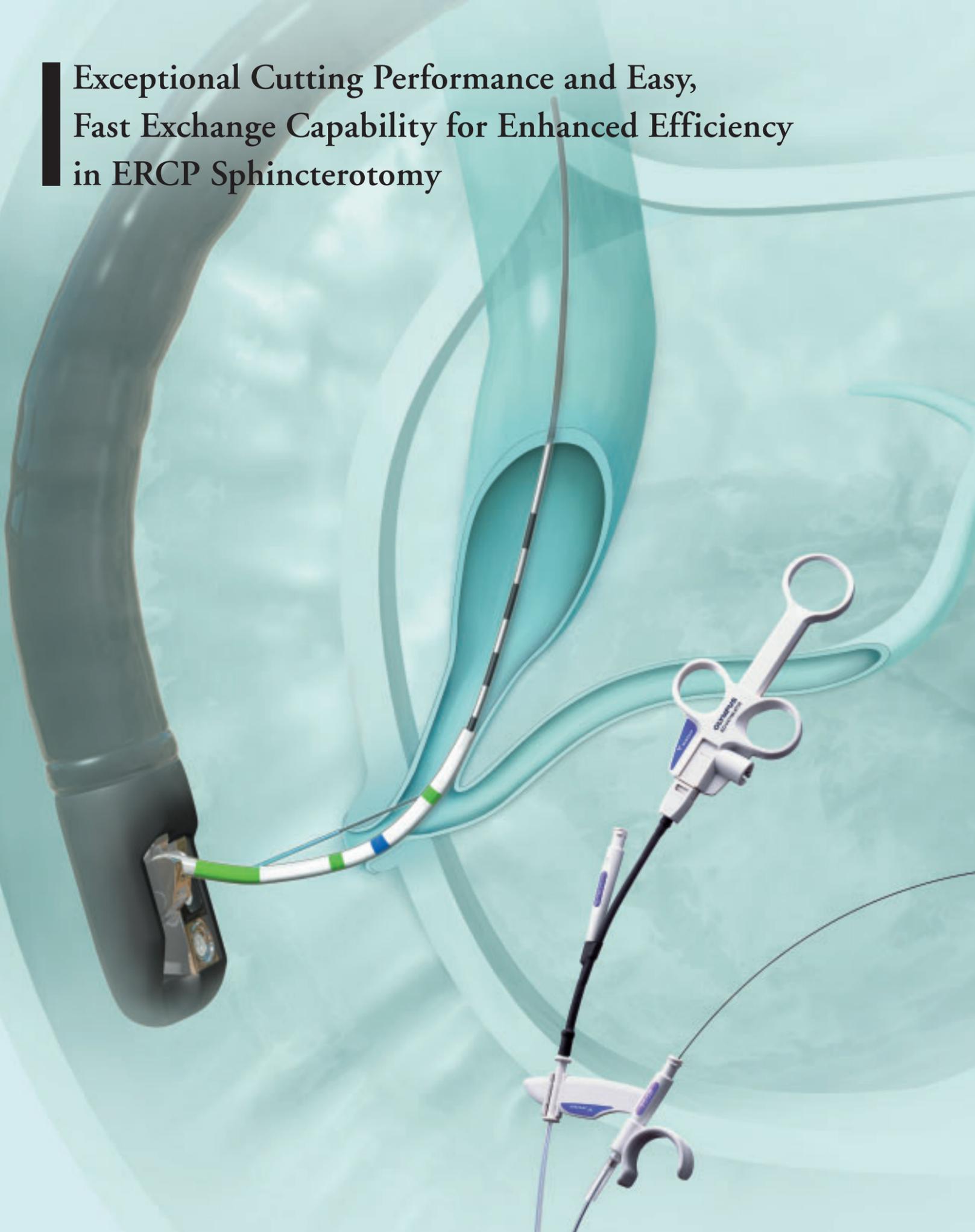

The Innovative V-System Design Lets You Proceed with Confidence and Efficiency


The V-System is a complete system that integrates Olympus endoscopes and EndoTherapy devices. The revolutionary V-System design offers the option of guidewire manipulation by the physician or the assistant, allows easier exchange of catheters, and enhances cannulation capability.



C-Hook

Now endoscopists have the option to manipulate guidewires and devices.


The convenient C-Hook allows the device handle to be attached to the endoscope's control section, putting it within easy reach of the endoscopist. With the device handle right at hand, the endoscopist can maneuver the guidewire, inject contrast media, and manipulate the handle — all while keeping a grip on the scope control section.

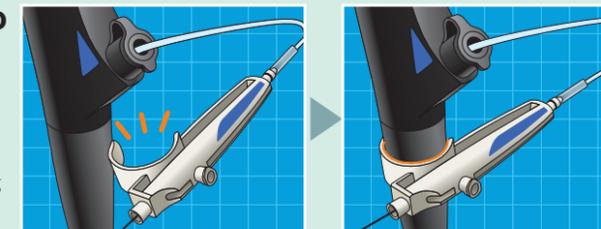

V-Marking

Indicates when to raise and lower the V-Groove forceps elevator.


The exclusive V-Marking is located on the proximal side of the sheath. When this marking reaches the channel port on the scope's control section, it indicates that the device tip has reached the distal end of the scope and the V-Groove forceps elevator may be lowered. When withdrawing the device from the scope, the same marking indicates when to raise the elevator to lock the guidewire.

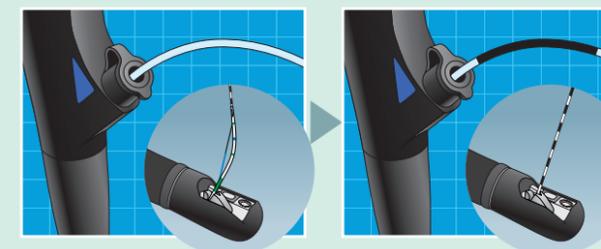

V-Sheath

Device control by the endoscopist or the assistant.

The V-Sheath allows the endoscopist complete device control or, if preferred, device control may be given to the assistant. The unique device design allows the guidewire sheath and injection sheath/handle to be separated. This forked sheath design allows either the endoscopist or the assistant to control the device.

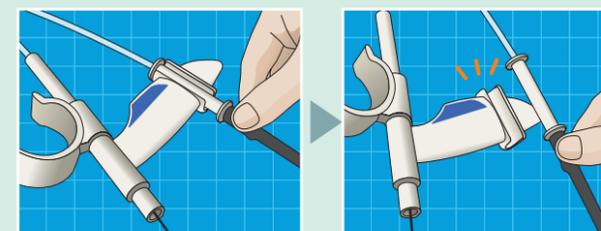

V-System device replacement procedure

Confirm the position of the V-Marking on the V-System EndoTherapy accessory.


When the V-Marking is completely visible above the instrument channel port, lift the forceps elevator to lock the guidewire.

The guidewire is now locked into the V-Groove.

Completely remove the device.



Exceptional Cutting Performance and Easy, Fast Exchange Capability for Enhanced Efficiency in ERCP Sphincterotomy

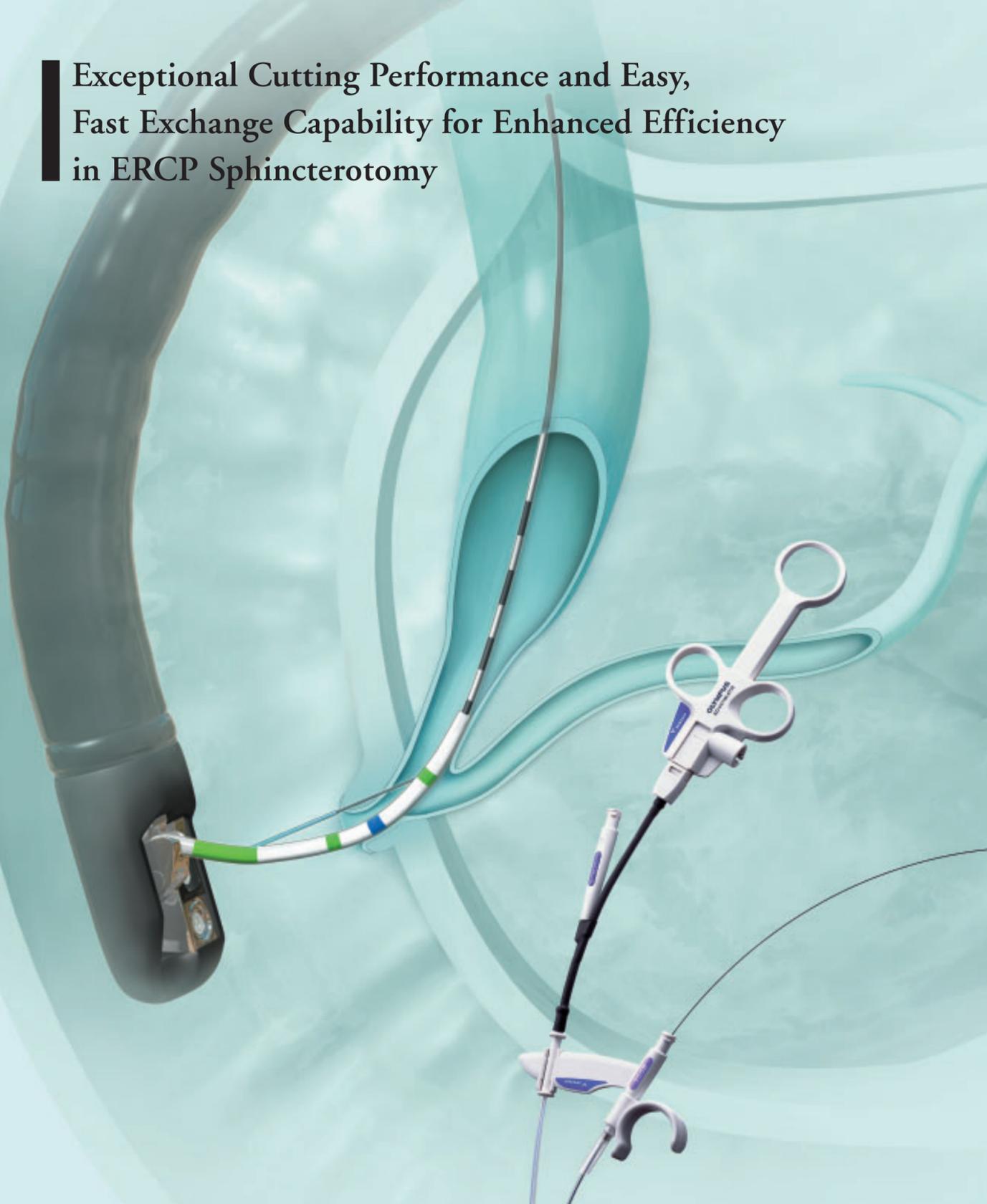


Unique Device Design and Attention to Every Detail of The CleverCut2V and CleverCut3V Sphincterotomes




CleverCut coating enhances safety

Olympus's signature CleverCut coating on the proximal end of the cutting wire minimizes damage to the surrounding tissue. In addition, CleverCut Coating reduces the risk of electrical contact between the wire and the endoscope.

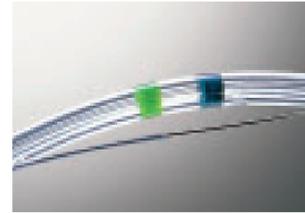


The CleverCut3V offers excellent orientation and smooth injection

The CleverCut3V wire, injection lumen and guidewire lumen are arranged to allow easier orientation of the cutting wire for effective sphincterotomy. Since the injection lumen and the guidewire lumen are completely separate, contrast media can be smoothly injected with a guidewire in place.

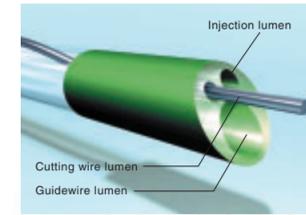


The CleverCut2V provides efficient cannulation capability

The CleverCut2V has two stiffening wires to provide stable cannulation and orientation.

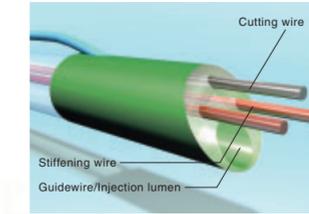



Pre-curved distal end for easier knife positioning

The distal ends of the CleverCut2V and CleverCut3V are pre-curved to achieve stable cannulation capability. This distal configuration also facilitates easy positioning of the knife into the papilla.

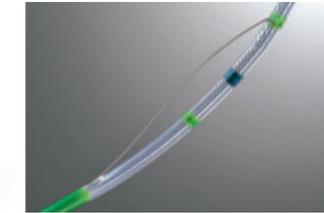


Distal marking on the sheath for improved view field visibility

The distal marking on the sheath clearly indicates both the center and cutting position of the knife.

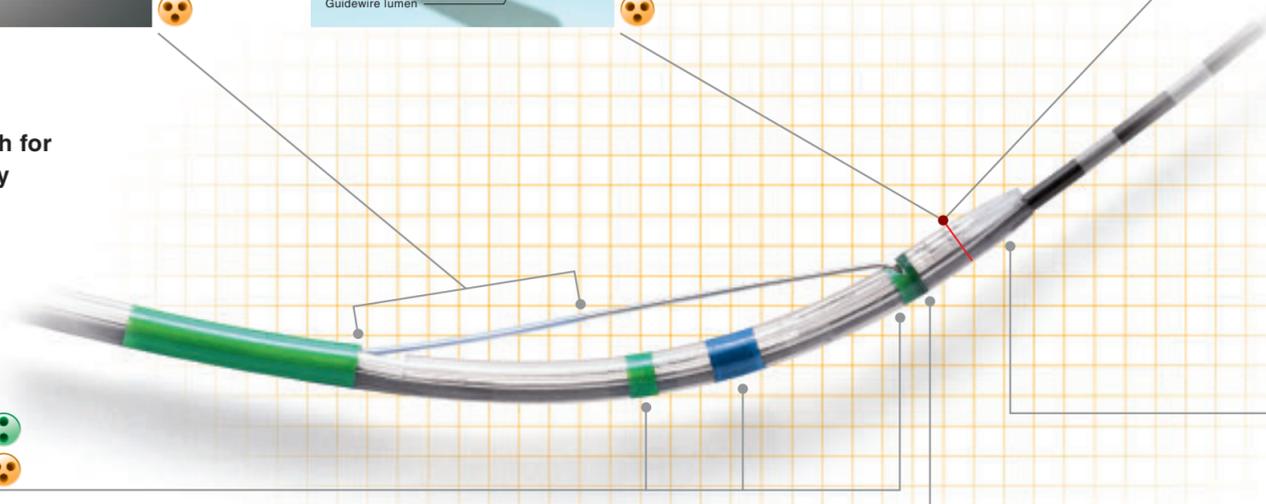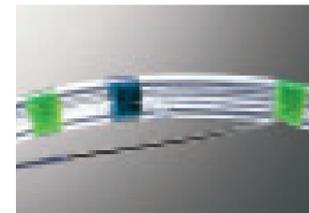

Tapered tip design for smooth insertion into strictures and the minor papilla (KD-V431Q series only)

The tapered tip design is ideally suited for cases in which cannulation is difficult due to strictures or when insertion into the minor papilla is required. The tapered tip CleverCut3V is compatible with a 0.025" diameter guidewire.

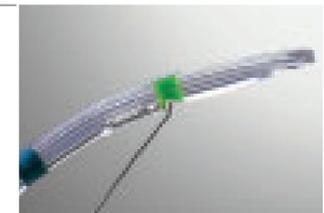

Easy identification of ports

The guidewire port and the injection port are easily identified by symbols.



Sheath design for stable and reliable cannulation

Designed to optimize insertion into the scope, this sheath is narrower at the distal end and thicker at the proximal end. This improves handling and ensures smoother insertion, while also providing excellent cannulation capability into the papilla.



Radiopaque tip markings for optimal visibility under fluoroscopy

The radiopaque tips of the CleverCut2V and CleverCut3V provide excellent visibility under fluoroscopy.

Single-use design for use-and-dispose convenience

The CleverCut2V and CleverCut3V are designed for single use only.



Specifications

Model	Lumen type	Distal tip diameter	Maximum insertion portion diameter	Working length	Compatible guidewire	Tip length	Knife length					
KD-V411M-0320	3-Lumen type 	4.5Fr	2.5mm	1700mm	0.89mm(0.035inch)	3mm	20mm					
KD-V411M-0330						3mm	30mm					
KD-V411M-0720						7mm	20mm					
KD-V411M-0725						7mm	25mm					
KD-V411M-0730						7mm	30mm					
KD-V411M-1520						15mm	20mm					
KD-V411M-1530						15mm	30mm					
KD-V411M-3030						30mm	30mm					
KD-V431M-0720						4.0Fr				0.64mm(0.025inch)	7mm	20mm
KD-V431M-0730											7mm	30mm
KD-V211M-0720	2-Lumen type 	4.5Fr	2.1mm		0.89mm(0.035inch)	7mm	20mm					
KD-V211M-0725						7mm	25mm					
KD-V211M-0730						7mm	30mm					
KD-V211M-1520						15mm	20mm					
KD-V211M-1530						15mm	30mm					
KD-V211M-3030						30mm	30mm					

Specifications, design and accessories are subject to change without any notice or obligation on the part of the manufacturer.

OLYMPUS[®]

OLYMPUS CORPORATION
Shinjuku Monolith, 3-1 Nishi-Shinjuku 2-chome, Shinjuku-ku, Tokyo 163-0914, Japan
OLYMPUS EUROPA GMBH
Postfach 10 49 08, 20034 Hamburg / Wendenstrasse 14-18, 20097 Hamburg, Germany
OLYMPUS AMERICA INC.
2 Corporate Center Drive, Melville, N.Y. 11747-3157, U.S.A.
OLYMPUS LATIN AMERICA, INC.
6100 Blue Lagoon Drive, Suite 390 Miami, Florida 33126-2087, U.S.A
KEYMED LTD.
Keymed House, Stock Road, Southend-on-Sea, Essex SS2 5QH, England

OLYMPUS SINGAPORE PTE LTD.
491B River Valley Road #12-01/04 Valley Point Office Tower, Singapore 248373
OLYMPUS HONG KONG AND CHINA LIMITED
Room 1520-1527, Ocean Centre, 5 Canton Road, Tsimshatsui, Kowloon, Hong Kong
OLYMPUS BEIJING REPRESENTATIVE OFFICE
Room 1401, East Tower, Gong Yuan No6 Royal Palace, No6 Gong Yuan Xijie, JianGuoMen Nei, DongCheng District, Beijing, 100005, China
OLYMPUS MOSCOW LIMITED LIABILITY COMPANY
117071, Moscow, Malaya Kaluzhskaya 19, bld. 1, fl.2, Russia
OLYMPUS AUSTRALIA PTY LTD.
31 Gilby Road, Mount Waverley, VIC., 3149, Australia

www.olympus.com

Printed in Japan F1082SB-3-0804