


Specifications

Model	Upper GI		Lower GI		Small intestines
	HX-201LR-135 A/B	HX-201LR-135L A/B	HX-201UR-135 A/B	HX-201UR-135L A/B	HX-201YR-135
Working length	1,650 mm		2,300 mm		2,700 mm
Quantity	A set : 20 pieces B set : 5 pieces		A set : 20 pieces B set : 5 pieces		5 pieces
Compatible channel	2.8 mm		2.8 mm		2.8 mm
Maximum diameter	2.6 mm		2.6 mm		2.6 mm
Clip length	Standard	Long	Standard	Long	Standard
Clip angle	135°		135°		135°

Intended Use

This instrument has been designed to be used with an Olympus endoscope for endoscopic clip placement within the gastrointestinal (GI) tract for the purpose of

- (1) endoscopic marking,
 - (2) hemostasis for
 - (a) mucosal / sub-mucosal defects < 3cm,
 - (b) bleeding ulcers,
 - (c) arteries < 2mm,
 - (d) polyps < 1.5cm in diameter,
 - (e) diverticula in the colon,
 - (3) as a supplementary method, closure of GI tract luminal perforations < 20mm that can be treated conservatively.
- Do not use these instruments for any purpose other than their intended use.

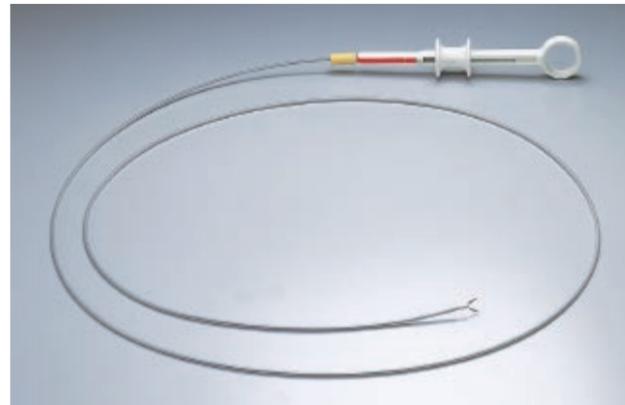

QuickClip2™/QuickClip2 Long™: Operating procedure

1 Once the device is seen at the distal end of the endoscope, remove the red stopper. The red stopper prevents the clip from extending out of the plastic sheath during insertion. Pull the yellow tube joint towards the handle to extend the clip from the plastic sheath.

2 Before approaching the targeted bleeding site, it is important to open the clip to its maximum width. To do this, gently pull the slider towards the thumb ring.

3 While holding the yellow tube joint, turn the handle slowly until the clip is in the proper position for clipping the target site.

4 While pressing the clip against the target site, pull the slider towards the thumb ring to grasp the target site deploy the closed clip. Remove the device from the endoscope.

Specifications, design and accessories are subject to change without any notice or obligation on the part of the manufacturer.

OLYMPUS

OLYMPUS MEDICAL SYSTEMS CORP.
Shinjuku Monolith, 3-1 Nishi-Shinjuku 2-chome, Shinjuku-ku, Tokyo 163-0914, Japan
OLYMPUS MEDICAL SYSTEMS EUROPA GMBH
Postfach 10 49 08, 20034 Hamburg / Wendenstrasse 14-18, 20097 Hamburg, Germany
OLYMPUS AMERICA INC.
3500 Corporate Parkway, P.O. Box 610, Center Valley, PA 18034-0610, U.S.A.
OLYMPUS LATIN AMERICA, INC.
5301 Blue Lagoon Drive, Suite 290 Miami, Florida 33126-2097, U.S.A.
OLYMPUS KEYMED GROUP LTD.
Keymed House, Stock Road, Southend-on-Sea, Essex S82 5QH, England
OLYMPUS SINGAPORE PTE LTD.
491B River Valley Road #12-01/04, Valley Point Office Tower, Singapore 248373

OLYMPUS HONG KONG AND CHINA LIMITED
L43, Office Tower, Langham Place, 8 Argyle Street, Mongkok, Kowloon, Hong Kong
OLYMPUS (BEIJING) SALES & SERVICE CO., LTD.
Room 1202, NCI Tower, A12 Jianguomenwai Avenue Chaoyang District Beijing, 100022, China
OLYMPUS MOSCOW LIMITED LIABILITY COMPANY
117071, Moscow, Malaya Kaluzhskaya 19, bld. 1, 1/2, Russia
OLYMPUS AUSTRALIA PTY LTD
31 Gilby Road, Mount Waverley, VIC., 3149, Australia
OLYMPUS KOREA CO., LTD.
4F, 5F, Gyeongnam Bldg., 157-27, Samsong-dong, Gangnam-ku, Seoul, Korea

www.olympus.com

OLYMPUS

Your Vision, Our Future

OLYMPUS
EndoTherapy
BEYOND THE SCOPE

A versatile product line and rotatable mechanism provide a wider range of hemostatic options than ever before




Single Use Rotatable Clip Fixing Devices

QuickClip2™

QuickClip2 Long™

Already featuring models with exceptional rotation abilities, Olympus's versatile line of QuickClip2™ and QuickClip2 Long™ devices now includes new models dedicated to the small intestine

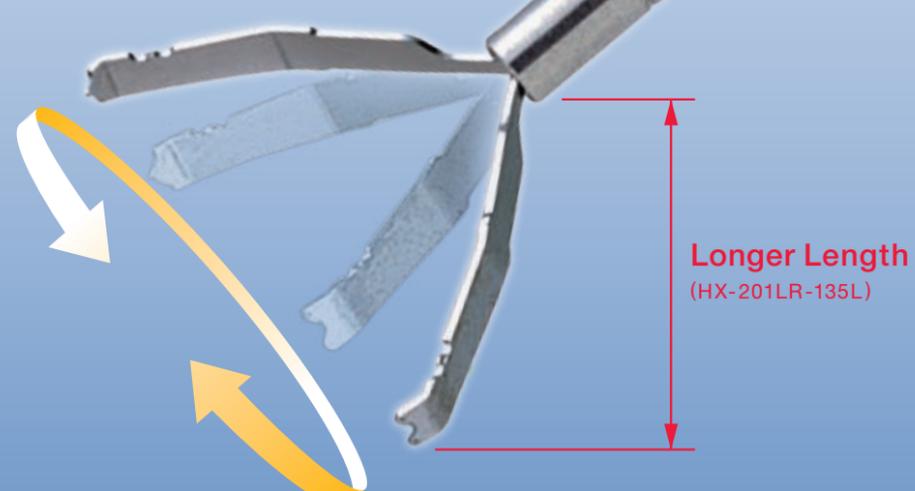

As the creator of endoscopic clipping — widely regarded as one of the most reliable hemostatic techniques in use today, Olympus is committed to maximizing the efficacy of this powerful technique. Our versatile line of QuickClip2™ already includes a proprietary design for smooth and consistent rotation as well as models with various opening widths for differing clinical needs. Now we've added models dedicated to the small intestine, giving you an even wider range of choices and ensuring your hemostatic clipping procedures will be fast and effective no matter what the situation.




Convenient rotation mechanism for easier targeting

The QuickClip2™ incorporates an exclusive rotation mechanism that enables you to more easily orient and position the clip at the site.


Longer clip length for expanded clearance

As the name suggests, the QuickClip2 Long™ is longer than its predecessor, the original QuickClip2™. The new design gives a much greater opening width and enables more tissue to be grasped at one time — a perfect compliment for the QuickClip2™ family.

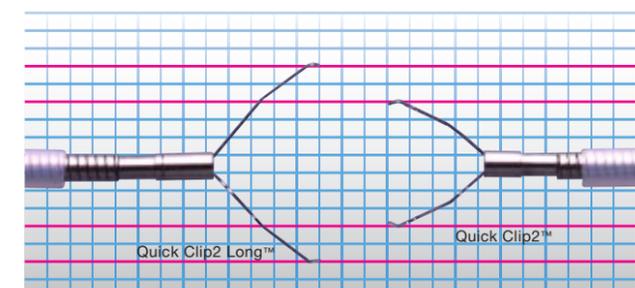

Simple and reliable hemostasis

Endoscopic clipping, considered to be one of the simplest and most reliable hemostatic techniques, mechanically binds blood vessels, minimizing the risk of re-bleeding and damage to the surrounding tissue, while providing instant visual feedback to confirm placement.

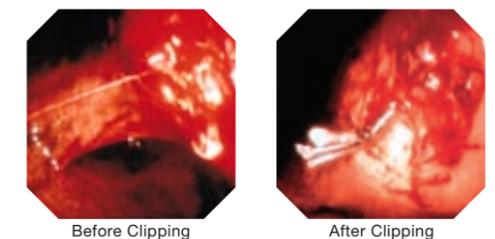

Wider, more versatile lineup

The QuickClip2™ and QuickClip2 Long™ lineup features models with a rotation mechanism designed for use in the upper gastrointestinal tract and models with longer prongs for use in the colon. Now the lineup has been expanded with the addition of new models specifically for use in the small intestine.