

OLYMPUS[®]

Your Vision, Our Future

Disposable Distal Attachment

**D-201-10704/11304/11802/
11804/12402/12704/13404/
14304/15004/16403**

OLYMPUS
EndoTherapy
BEYOND THE SCOPE

**Ideal for Maintaining the Field of View
in Endoscopic Examination and Treatment**




Easier Viewing of the Site, Easier Manipulation of the Device



● How to mount the distal attachment

1. Orient the distal attachment so that the side hole is aligned with the endoscope's objective lens.
2. Push the endoscope tip into the distal attachment as far as the alignment line on the distal attachment.

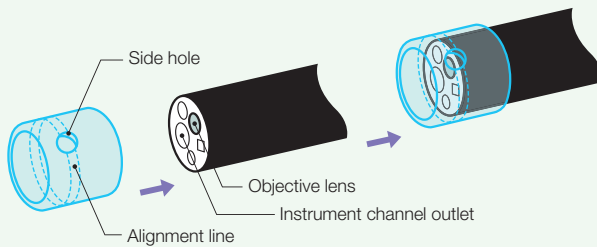

● How to drain fluid

When fluid pooled in the distal attachment blocks the endoscopic field of view, bring the side hole of the attachment in contact with tissue. This facilitates expelling of the fluid.

Features

● Maintains the optimal field of view

By maintaining the appropriate distance between the endoscope tip and observation site, the distal attachment helps maintain the optimal field of view. The addition of a convenient side hole on the sleeve of the 4 mm working distance models ensures that any fluid inside can be drained at all times.

● Rounded edge is easy on tissue

The edge of the distal attachment has a round shape to reduce damage to tissue.

● Versatile lineup for Olympus GI endoscopes

Ten different distal attachments are available, ensuring that you can find one to match the outer diameter of your Olympus upper or lower endoscope.

● Models dedicated to magnification endoscopes

Three of the distal attachment models are specifically designed for use with magnification endoscopes, making it easier to keep an image in focus and achieve correct depth of field.

● Sterile, single-use, soft design

Specifications

4 mm working distance type

Model	Outer diameter	Length from distal end of endoscope	Compatible endoscopes
D-201-10704	11.35 mm	4.0 mm	GIF-XQ140/Q165/ XQ230/XQ240/ SP240/XQ260/Q260/V70/ SIF-Q260/SIF-Q180
D-201-11304	11.8 mm	4.0 mm	GIF-XQ40/V/130/140/Q160/ Q240X/E3/Q145/H260
D-201-11804	12.4 mm	4.0 mm	GIF-Q140/H180/Q230/Q240/ Q260J/PCF-P240A/Q260J
D-201-12704	13.4 mm	4.0 mm	GIF-Q40/PCF-140/Q150A/160A/ 1TQ160/230/240/Q260A/ Q180A/H180A
D-201-13404	14.0 mm	4.0 mm	GIF-2T240/2TQ260M/CF-240/ 240A/240D/Q260A/Q260D/ OSF-4/PCF-E3
D-201-14304	15.0 mm	4.0 mm	GIF-XTQ160/2T160/CF-V70/ 130/140L/140L/Q140/Q160/ Q160A/Q165/Q240/Q240A/V/ Q145/Q160S/Q180A/H260A
D-201-15004	15.7 mm	4.0 mm	CF-30/40/E3/Q160D/H180A/ H180D/2T160/230/H260D

10 pieces/pack

Magnification endoscope dedicated type

Model	Outer diameter	Length from distal end of endoscope	Compatible endoscopes
D-201-11802	12.1 mm	2.0 mm	GIF-Q240Z
D-201-12402	12.7 mm	2.0 mm	GIF-Q160Z/H260Z/FQ260Z
D-201-16403	16.7 mm	3.0 mm	CF-Q160Z/Q240Z/FH260AZ

12 pieces/pack

Specifications, design and accessories are subject to change without any notice or obligation on the part of the manufacturer.

OLYMPUS[®]

OLYMPUS MEDICAL SYSTEMS CORP.
Shinjuku Monolith, 3-1 Nishi-Shinjuku 2-chome, Shinjuku-ku, Tokyo 163-0914, Japan
OLYMPUS MEDICAL SYSTEMS EUROPA GMBH
Postfach 10 49 08, 20034 Hamburg / Wendenstrasse 14-18, 20097 Hamburg, Germany
OLYMPUS AMERICA INC.
3500 Corporate Parkway, B.O. Box 610, Center Valley, PA 18034-0610, U.S.A.
OLYMPUS LATIN AMERICA, INC.
5301 Blue Lagoon Drive, Suite 290 Miami, Florida 33126-2097, U.S.A.
OLYMPUS KEYMED GROUP LTD.
Keymed House, Stock Road, Southend-on-Sea, Essex SS2 5QH, England
OLYMPUS SINGAPORE PTE LTD.
491B River Valley Road #12-01/04, Valley Point Office Tower, Singapore 248373

OLYMPUS HONG KONG AND CHINA LIMITED
L43, Office Tower, Langham Place, 8 Argyle Street, Mongkok, Kowloon, Hong Kong
OLYMPUS (BEIJING) SALES & SERVICE CO., LTD.
Room 1202, NCI Tower, A12 Jianguomenwai Avenue Chaoyang District Beijing, 100022, China
OLYMPUS MOSCOW LIMITED LIABILITY COMPANY
117071, Moscow, Malaya Kaluzhskaya 19, bld. 1, fl.2, Russia
OLYMPUS AUSTRALIA PTY LTD
31 Gilby Road, Mount Waverley, VIC., 3149, Australia
OLYMPUS KOREA CO., LTD.
4F, 5F, Gyeongang Bldg., 157-27, Samseong-dong, Gangnam-ku, Seoul, Korea

www.olympus.com