

OLYMPUS[®]



Suction Polyp Trap[™]

Simple and easy to use, the Suction Polyp Trap is a reliable solution to capture excised tissue.

Suction Polyp Trap™



Cat. No. H334 | Suction Polyp Trap | 12 Units/Case

Features and Benefits

- | | |
|-------------------------------------|--|
| Clear plastic construction | Allows visual verification of polyp collection. |
| Four-chamber indexing system | Permits accurate identification and localization of each polyp. |
| Four capture chambers | Offers convenient collection of up to four polyps or tissue specimens, isolating each for identification and pathology study. Reduces contact with patient effluent. |
| Easy in-line installation | Installs between the endoscope and the suction receptacle in only a few seconds. |
| Individually packaged | Reduces risk of biohazard and sample contamination with single-patient use. |
| Lab transport device | Provides easy and safe transport to pathology lab with the sealed unit. |

Installing the Suction Polyp Trap

- 1 Connect tube on center port of trap to suction system.
- 2 Connect outside port to endoscope.
- 3 Rotate jar to align arrow on lid with any open ("0") position for normal suctioning.
- 4 To entrap first polyp, rotate jar and align arrow with number 1.
- 5 After the polyp is trapped, repeat Step 3 for normal suctioning as needed.
- 6 Repeat Step 4 for positions 2, 3, and 4 for each additional polyp (same patient only).
- 7 Follow standard procedures to prepare specimen(s) or simply disconnect tubing from endoscope and suction systems; then remove center tube from Suction Polyp Trap lid.
- 8 Fill Suction Polyp Trap jar with fixative. Replace lid when fixative filling is complete.
- 9 Remove center tube from lid. Bend outer tube to cover both outer and center ports.
- 10 Unit is ready for transport to laboratory.

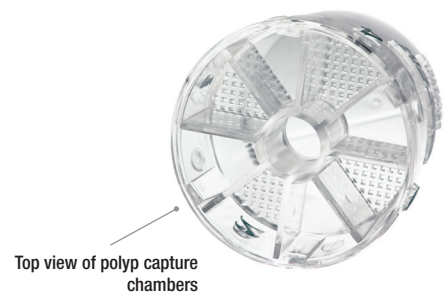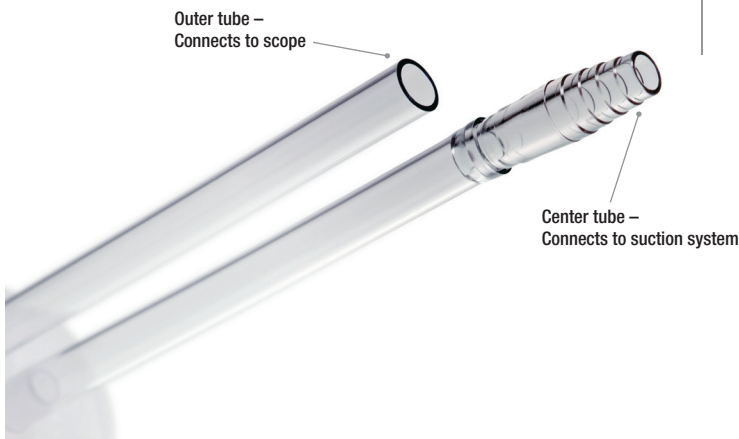

OAIET0110BR06223

To place an order, call **800-848-9024**.
www.olympusamerica.com/polyptrap

OLYMPUS
EndoTherapy

Manufactured by Bracco Diagnostics Inc.
 ©2010 Olympus America Inc. All rights reserved.
 Olympus and Olympus EndoTherapy are registered trademarks of Olympus Corporation, Olympus America Inc., and/or their affiliates. Suction Polyp Trap is a trademark of Bracco Diagnostics Inc.