



Your Vision, Our Future

Quick Catch Polyp Trap™

Easy installation, fast retrieval,
and efficient sample preparation,
the Quick Catch Polyp Trap is
the cost-effective choice to
capture excised tissue.



Quick Catch Polyp Trap™

Cat. No. H332 | Quick Catch Polyp Trap | 50 Units/Case

Features and Benefits

Simple in-line installation	Installs in only a few seconds between the endoscope and the suction receptacle.
Clear plastic construction	Allows visual verification of polyp collection.
Unique design	Provides flat surface for easy gripping and handling.
Sized to fit into specimen jar	Eliminates need for specimen handling and transfer.
Luer lock design on base	Allows syringe attachment to aid in sample removal.
Cost-effective	Offers money savings when polyp indexing and effluent bypass are not needed.
Individually packaged	Reduces risk of biohazard and sample contamination with single-patient use.
Dispenser pack	Allows easy identification and access to inventory.



Installing the Quick Catch Polyp Trap

Connection

- 1 Connect barbed fitting of trap to suction system.
- 2 Connect tube side of trap to endoscope suction fitting.

Capture

- 3 Apply suction via the endoscope suction valve to trap polyp.

Polyp Removal and Transport

- 4 After the polyp is trapped, remove the trap from the endoscope and suction system.
- 5 Separate trap by rotating both halves in opposite directions.
- 6 Remove polyp and place in specimen jar with fixative, or place trap in specimen jar and fill with fixative.
- 7 Transport to pathology.



OAIET0410BR0XXXX

To place an order, call **800-848-9024**.
www.olympusamerica.com/quickcatch

OLYMPUS

Endo**Therapy**®

Manufactured by Bracco Diagnostics Inc.
 © 2010 Olympus America Inc. All rights reserved.
 Olympus and Olympus EndoTherapy are registered trademarks of Olympus Corporation, Olympus America Inc., and/or their affiliates. Quick Catch is a trademark of Bracco Diagnostics Inc.