
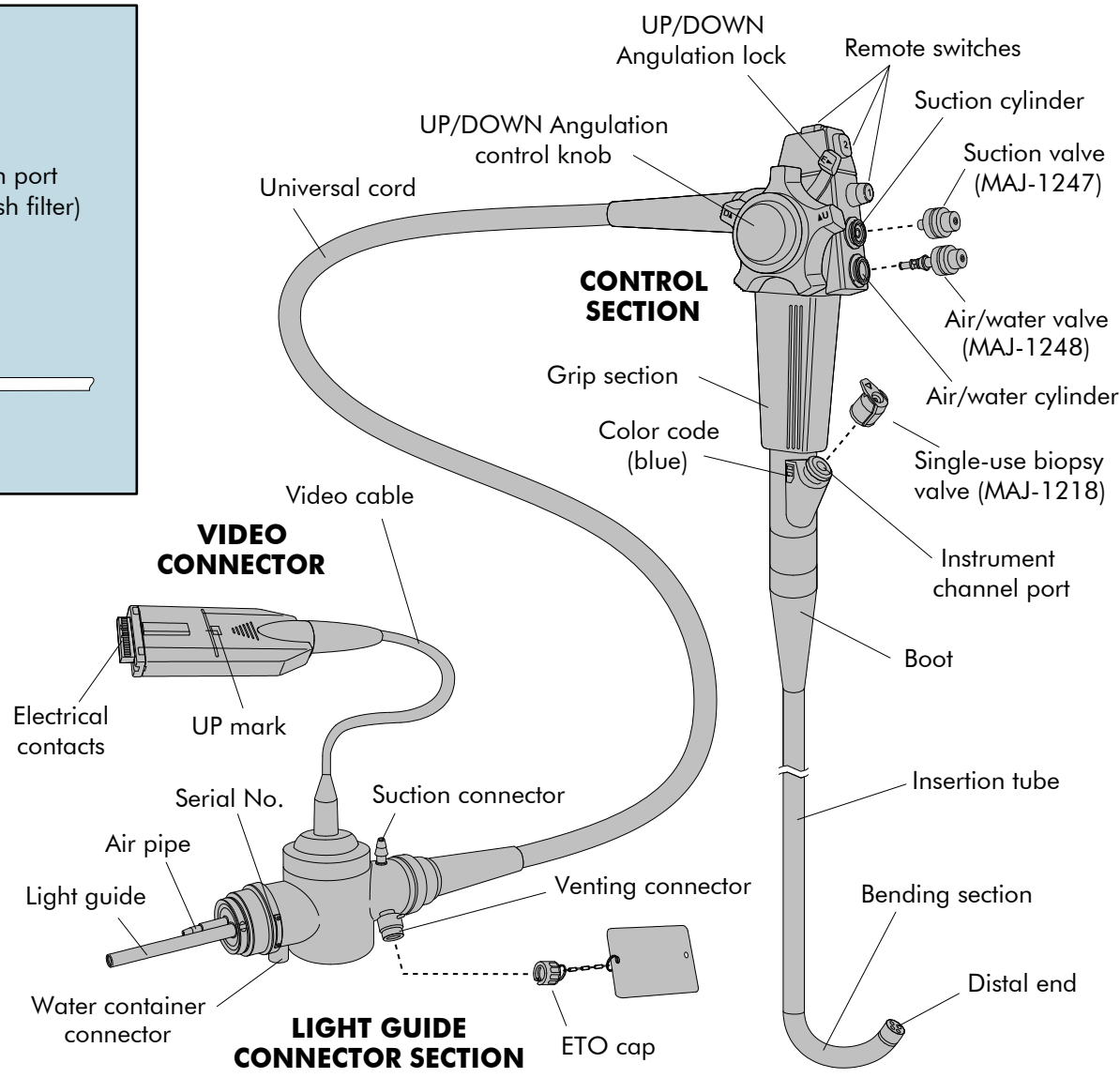
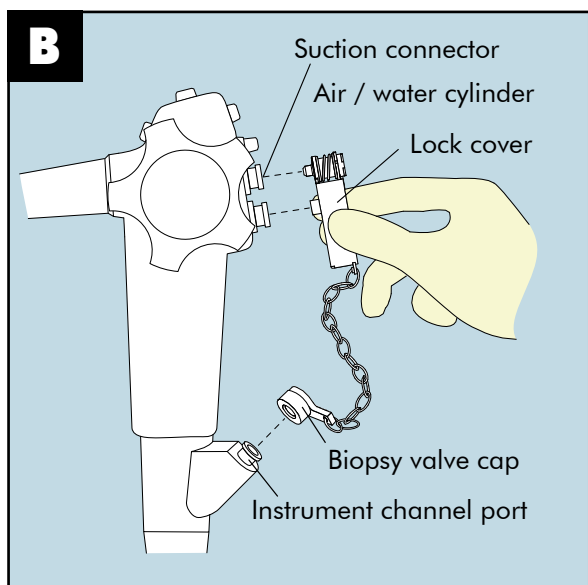


ENDOSCOPE CHANNEL GUIDE


VISERA™ PEF-V Transnasal Esophagovideoscope

WARNING: This guide is only a summary of the connections used for reprocessing your PEF-V esophagoscope. Be sure to follow the detailed steps outlined in the *INSTRUCTIONS* manual that was included with your endoscope when purchased.

CONNECTION OF INJECTION TUBE (A), CHANNEL PLUG (B) & NOMENCLATURE

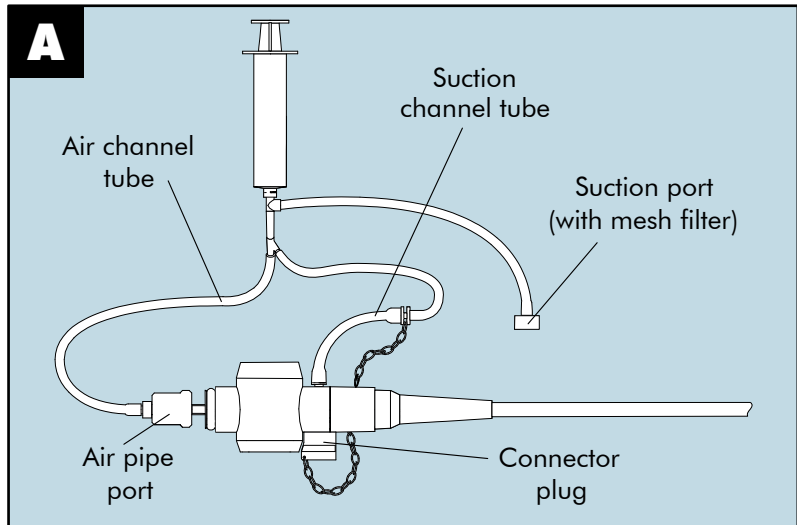

AIR, WATER & SUCTION CHANNEL SYSTEM

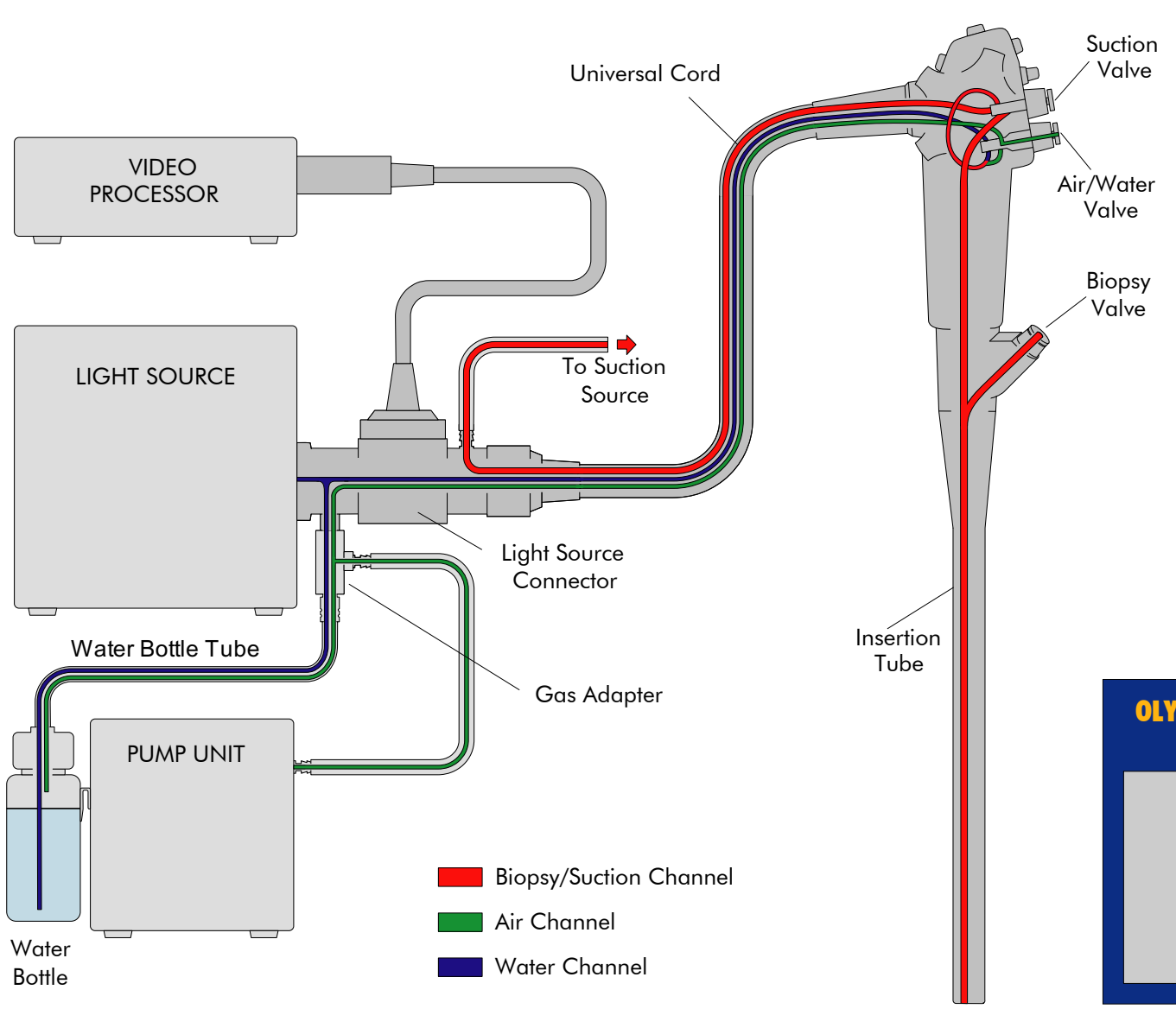

IMPORTANT

- Failure to properly clean and high-level disinfect endoscopic equipment after each examination can compromise patient safety.
- Every channel of the endoscope must be reprocessed each time the endoscope is used, even if the channel was not utilized during the preceding patient procedure.
- Every channel must go through every reprocessing step (cleaning, disinfection, rinsing, alcohol-air drying).
- If an automated reprocessor is used, make sure that all channels are being properly reprocessed. Follow all manufacturers' instructions.

OLYMPUS CUSTOMER SERVICE CENTER
800-548-5515

IF YOU HAVE ANY QUESTIONS,
CONTACT YOUR LOCAL
OLYMPUS REPRESENTATIVE