

ENDOSCOPE DISINFECTION GUIDE


VISERA™ PEF-V Transnasal Esophagovideoscope

— Refer to companion *ENDOSCOPE CLEANING GUIDE* for cleaning information —

WARNING: This guide is only a summary of the steps necessary to properly disinfect your endoscope. Be sure to follow the detailed steps outlined in the *INSTRUCTIONS* manual that was included with your endoscope when purchased.

HIGH-LEVEL DISINFECTION


1

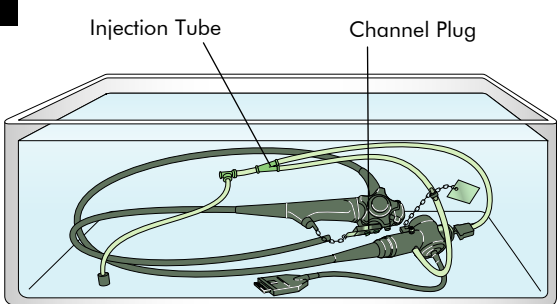

Injection Tube Channel Plug


Fill basin with disinfectant. Attach channel plug and injection tube to previously cleaned endoscope. Immerse completely in disinfectant solution.

2

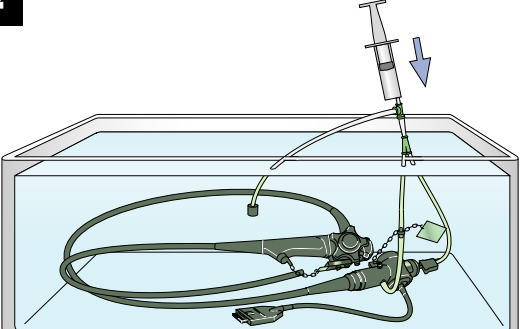


Use syringe to completely fill air, water and suction channels with disinfectant solution.

3

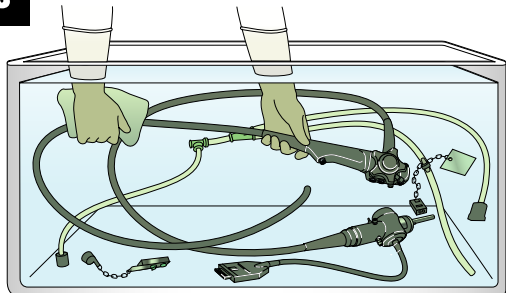

While the endoscope is immersed, disconnect cleaning accessories and use a clean lint-free cloth to wipe away any bubbles clinging to external surfaces.


4

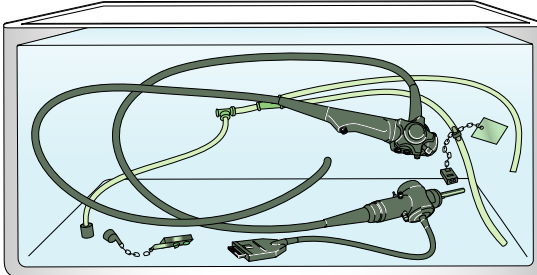

Soak all equipment according to the disinfectant manufacturer's recommended time and temperature. Use a timer to measure exact contact time.

RINSING


5

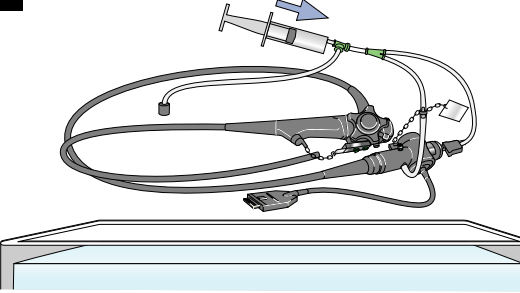

Connect channel plug and injection tube. Inject air to remove disinfectant from air, water and suction channels. Remove endoscope from disinfectant.


6

CAUTION: Follow disinfectant manufacturer's rinsing instructions.

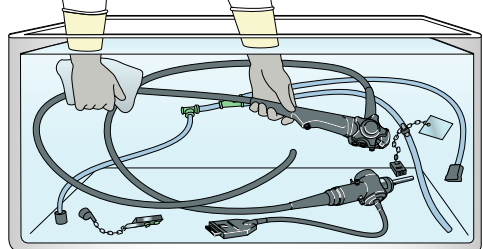

Disconnect channel plug and injection tube. Immerse endoscope and detached parts in water. Thoroughly rinse all external surfaces.


7

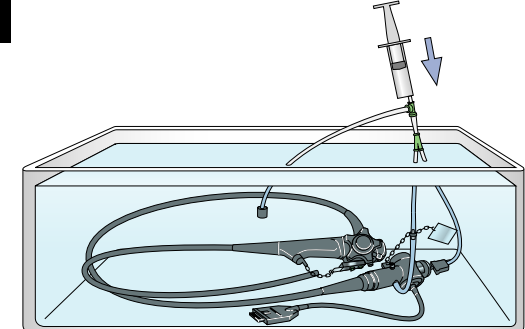

Connect channel plug and injection tube to endoscope. Use a syringe to flush water through air, water and suction channels.


DRYING

8

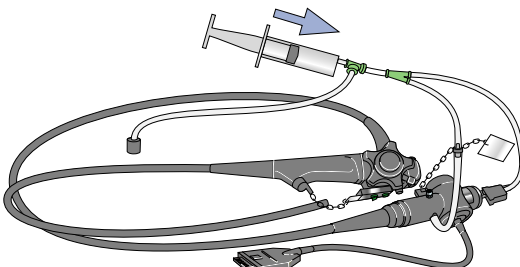

Remove endoscope from rinse water. Use syringe to inject air through air, water and suction channels, expelling the rinse water.


9

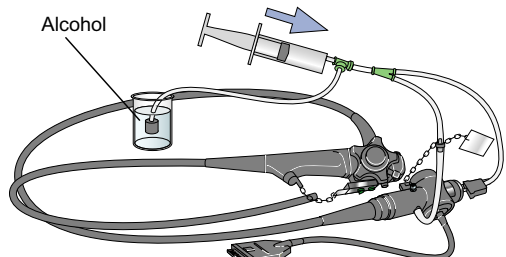

Alcohol


Use syringe to flush 70% alcohol through air, water & suction channels. Inject air to remove retained alcohol.

10

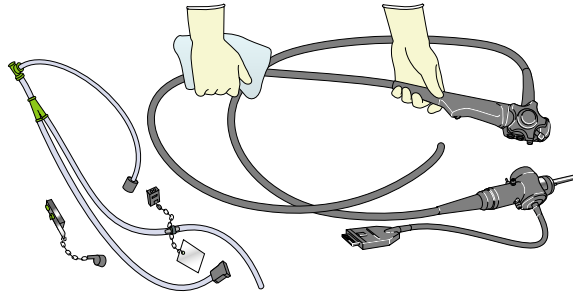

Disconnect all accessories from the endoscope. Dry endoscope and accessories with an alcohol-moistened cloth. Make sure equipment is completely dry prior to storage.

TRANSPORTATION

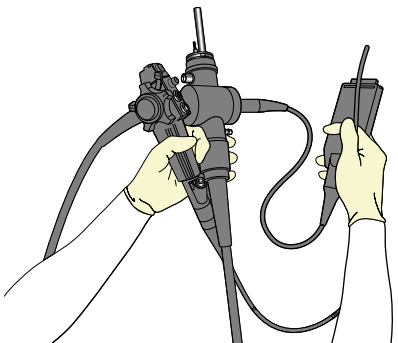

Hold all parts of the endoscope when carrying. Do not let the ends of the endoscope swing freely.

IMPORTANT

- Use a compatible disinfectant. Test for proper concentration per manufacturer's instructions.
- Autoclave all autoclavable accessories.
- Store endoscope hanging vertically with valves and caps removed, and with angulation lock in a "free" position.

Clean endoscope meticulously prior to disinfection.